

## System Ankle Joints – Pressing in Feather Keys for Joint Conversion

Some of our system ankle joints have so-called feather keys with pin which are located inside the joint's upper part. In the following we describe how to proceed if you have to mount the feather keys with pin yourself due to maintenance, joint conversion etc.

### FIOR & GENTZ

Gesellschaft für Entwicklung und Vertrieb  
von orthopädietechnischen Systemen mbH

Dorette-von-Stern-Straße 5  
21337 Lüneburg

☎ +49 4131 24445-0  
☎ +49 4131 24445-57

✉ [info@fior-gentz.de](mailto:info@fior-gentz.de)  
🌐 [www.fior-gentz.de](http://www.fior-gentz.de)







Clean the grooves of the joint's upper part **thoroughly**, otherwise the feather keys with pin cannot be mounted correctly. This is important, as incorrectly mounted feather keys with pin are very hard to remove.

## Step 2/6



Place both feather keys with pin into the openings of the assembly/lamination dummy.

### **FIOR & GENTZ**


Gesellschaft für Entwicklung und Vertrieb  
von orthopädietechnischen Systemen mbH

Dorette-von-Stern-Straße 5  
21337 Lüneburg

☎ +49 4131 24445-0  
☎ +49 4131 24445-57


✉ info@fior-gentz.de  
🌐 www.fior-gentz.de

**FIOR & GENTZ**  
ORTHOPÄDIETECHNIK MIT SYSTEM

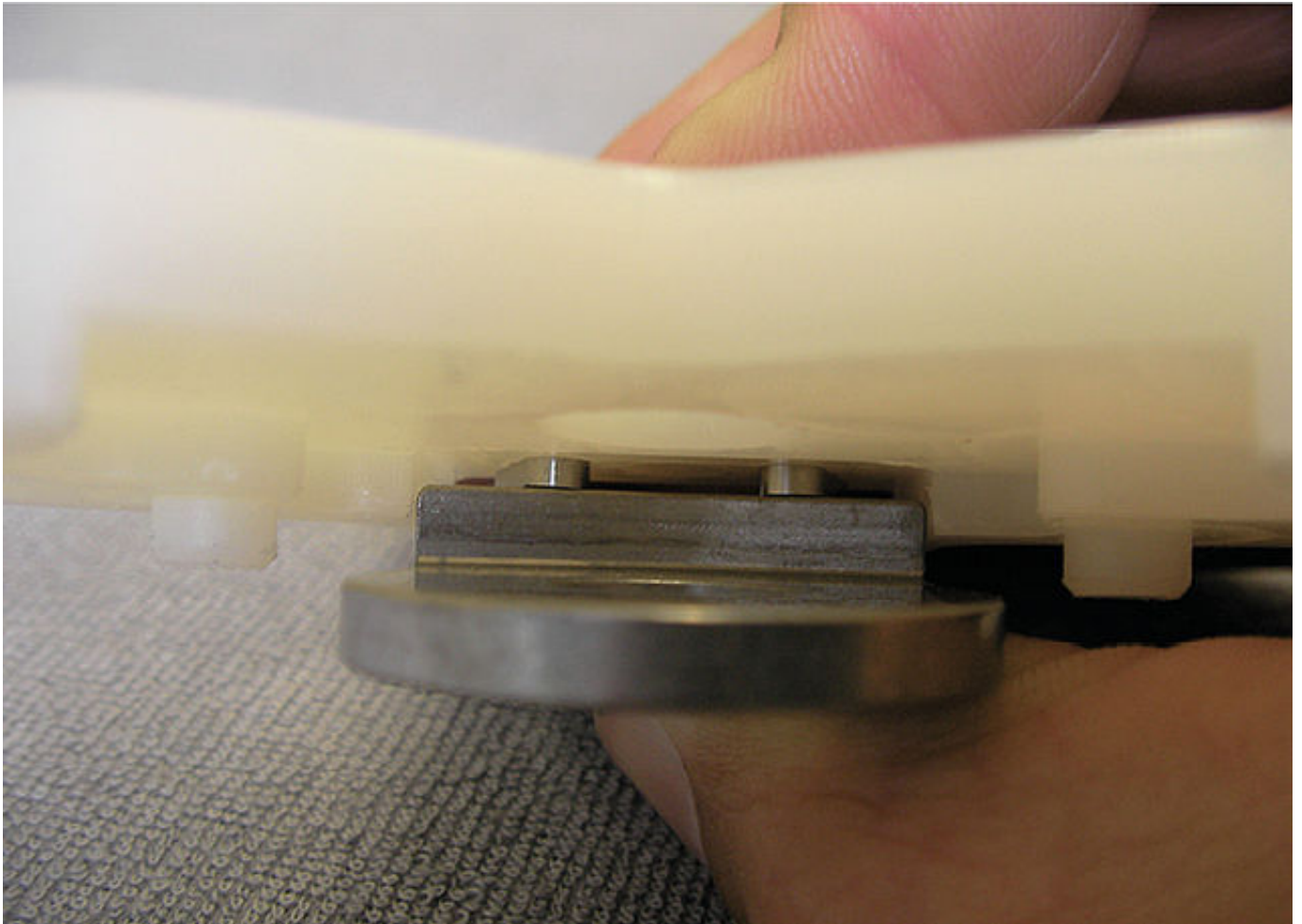


Place the assembly/lamination dummy onto the joint's upper part in such a way that the feather keys with pin are inserted into the grooves.



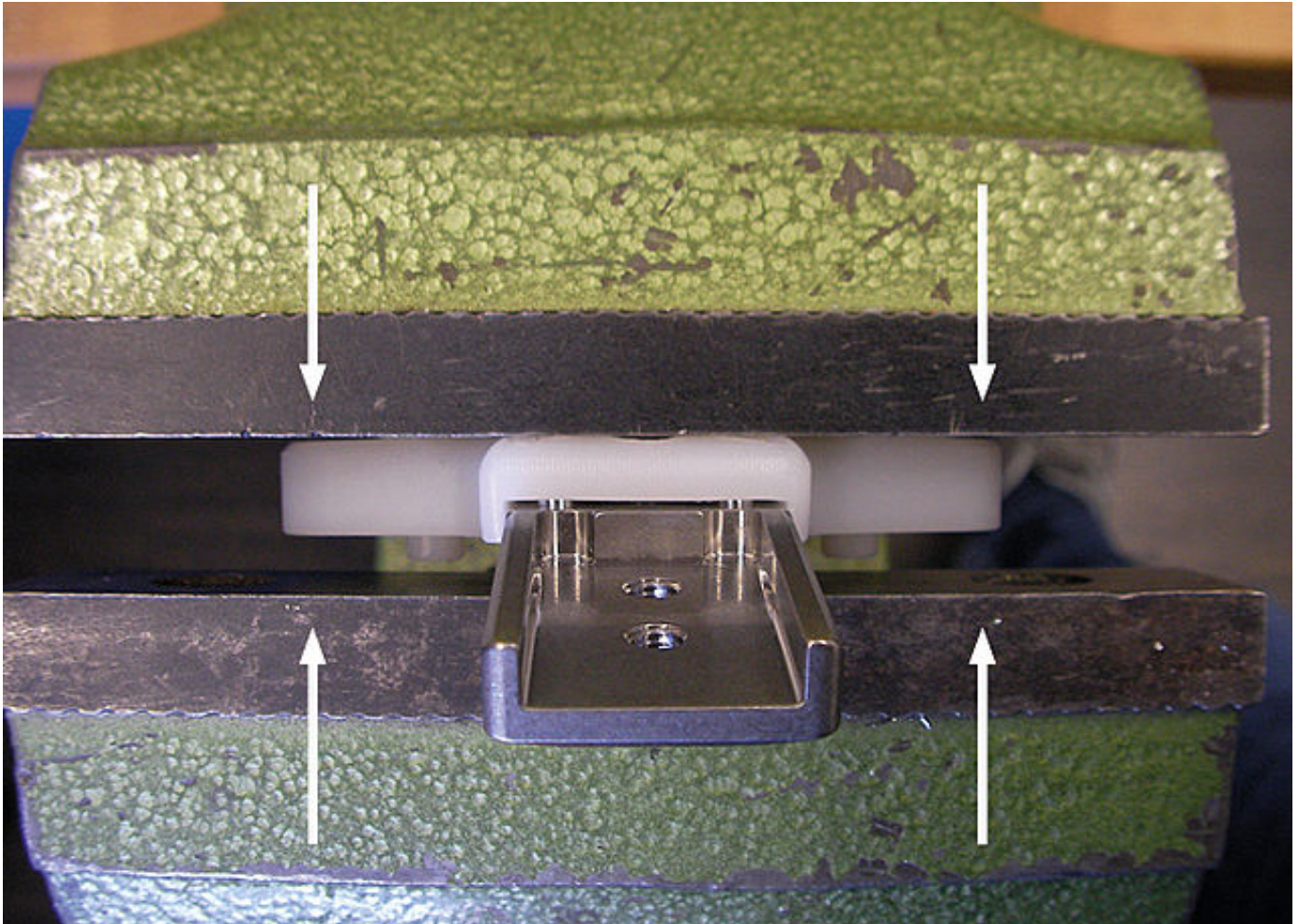

Press the assembly/lamination dummy to the joint's upper part in a vice with plain jaws until there is no opening left between them.

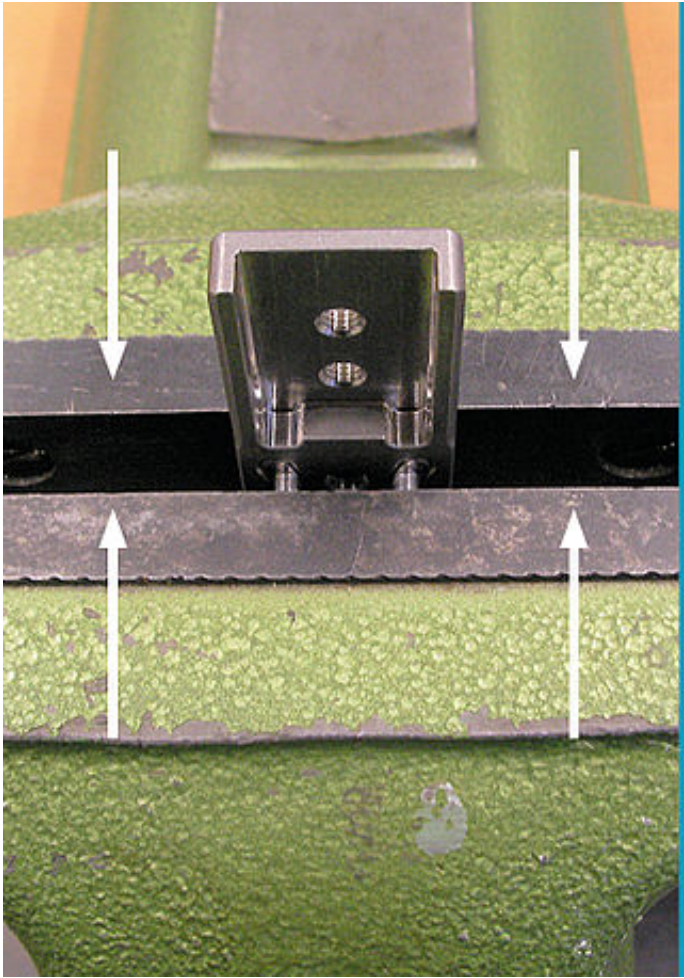

Remove the assembly/lamination dummy. Press the feather keys with pin carefully to the joint's upper part in a vice with plain jaws until the feather keys with pin are flush with the joint's upper part.



Mount the cover plate to the joint's upper part **without** adhering them. Press the joint's upper part to the cover plate as described in the [manual](#). Continue assembling the joint as usual.



## FIOR & GENTZ

Gesellschaft für Entwicklung und Vertrieb von orthopädietechnischen Systemen mbH

Dorette-von-Stern-Straße 5  
D-21337 Lüneburg

Tel.: [+49 4131 24445-0](tel:+494131244450)  
Fax: [+49 4131 24445-57](tel:+4941312444557)  
E-Mail: [info\(at\)fior-gentz.de](mailto:info@fior-gentz.de)

## Beratung und Technischer Support

[Verkauf im Innendienst](#)

[Verkauf im Außendienst](#)

[Technischer Support](#)

[Export](#)



### FIOR & GENTZ

Gesellschaft für Entwicklung und Vertrieb  
von orthopädietechnischen Systemen mbH

Dorette-von-Stern-Straße 5  
21337 Lüneburg

☎ [+49 4131 24445-0](tel:+494131244450)  
☎ [+49 4131 24445-57](tel:+4941312444557)

✉ [info@fior-gentz.de](mailto:info@fior-gentz.de)  
🌐 [www.fior-gentz.de](http://www.fior-gentz.de)

**FIOR & GENTZ**  
ORTHOPÄDIETECHNIK MIT SYSTEM

