Maintenance Carbon System Ankle Joints

The NEURO SWING H₂O system ankle joint has to be checked for wear and functionality every six months during maintenance. Here, we give you a detailed overview of the work steps to be taken at every maintenance.

You will find further information on the system ankle joint in the [instructions for use](#).

Furthermore, you can find the online tutorial for the joint assembly of the carbon system ankle joints [here](#).

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✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Step 1/11

You can find information on system components that have to be checked particularly carefully during maintenance in the instructions for use under “Maintenance”.

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☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Demount the spring units in order to examine it. Check if the system joint moves without lateral play.

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✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Step 3/11

Hold the tibial shell and try to move the foot shell in pronation-supination direction.
If there is lateral play, this can indicate loosened screws and/or worn out sliding washers.

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☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Check if there is play around the axis. If there is play, the bearing nut and the sliding bushing should be replaced if necessary.

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✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Step 5/11

Check if the system joint moves freely. Bring the system joint in plantar flexion/dorsiflexion. If the system joint does not move freely, mount the next thinner sliding washer or clean the system joint thoroughly.

Note: At a bilateral orthosis, only check one system joint at a time.

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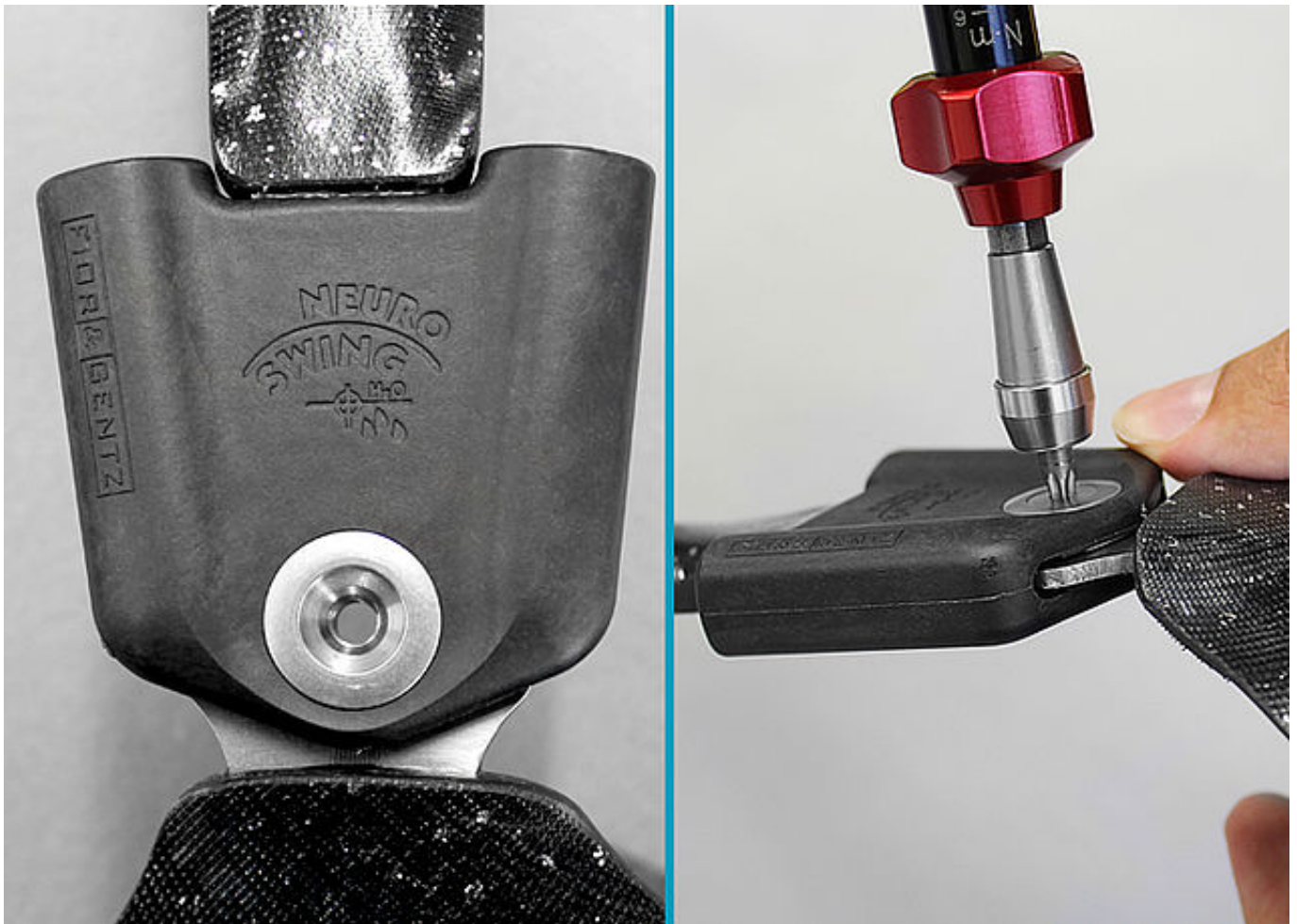
☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Disassemble the system joint completely. Clean all component parts and check them carefully.

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☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

- damaged sliding washer
- worn out bearing nut
- worn out sliding bushing
- worn out screw

Replace component parts if they are worn out.

Replace the sliding bushing if it is worn out.

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☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Replace the spring unit in case of loud noise generation and/or a reduced spring force or a limited range of motion.

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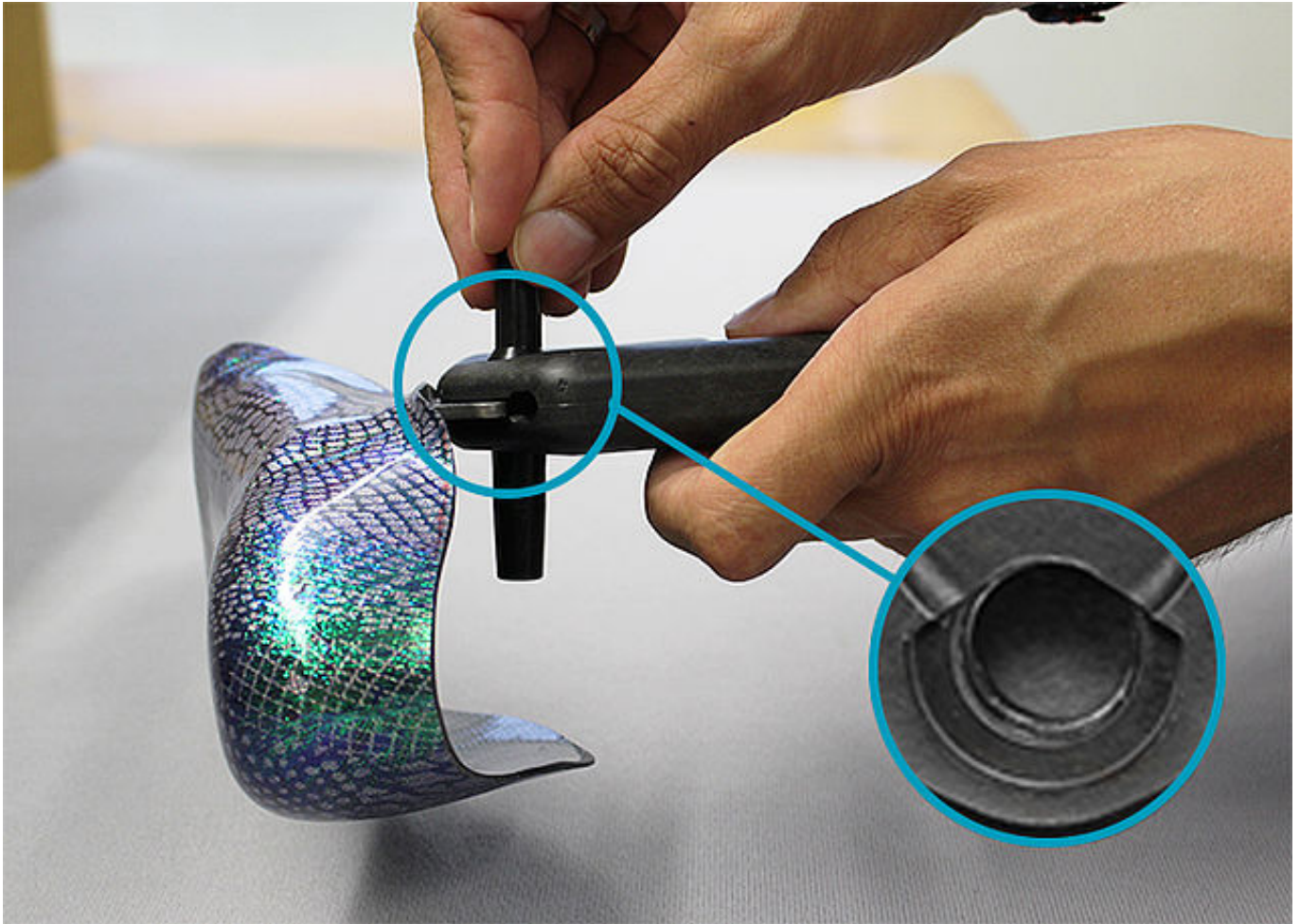
☎ +49 4131 24445-0
☎ +49 4131 24445-57

✉ info@fior-gentz.de
🌐 www.fior-gentz.de

Check the spring unit's O-ring. Replace the O-ring if it is worn out.

Note the online tutorial for the joint assembly of the carbon system ankle joints in order to reassemble the system joint.

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