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Content	Page
1. CE Conformity	13
2. Warranty	13
3. Safety Information	13
4. Safety Instructions	13
4.1 Classification of the Safety Instructions	13
4.2 Structure of the Safety Instructions	14
4.3 All Instructions for a Safe Handling of the P1 Foot Lifter Orthosis	14
5. Application	16
6. Functionality	16
7. Scope of Delivery	16
8. Choose the Right Orthosis Size	16
9. Tools for Assembling the P1 Foot Lifter Orthosis	16
10. Adapting and Processing the P1 Foot Lifter Orthosis	17
10.1 Carbon Fibre Sole	17
10.2 Spring Bar	18
10.3 Heat Shrink Tube	19
10.4 Calf Band	19
10.5 Hook and Loop Fastener and Pad	19
10.6 Padding Sole	20
11. Putting the P1 Foot Lifter Orthosis on	20
12. Cleaning	20
13. Spare Parts	21
14. Information for the Patient's Documentation	23

1. CE Conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of Regulation (EU) 2017/745. Therefore, the FIOR & GENTZ products bear the CE marking.

Printed version: 2020-03

2. Warranty

The warranty applies only if the product is used under the described conditions and for the intended purpose. Furthermore, we do not accept a multi-use of the product. Other than that, we refer to our General Terms and Conditions of Business Transactions, Sales, Delivery and Payment.

3. Safety Information

These instructions for use for qualified personnel are addressed to orthotists. That is why the content basically confines to features of the product. It does not contain any notes about dangers which are obvious to orthotists.

For getting a maximum of safety, please show the patient and/or the care team how to use and maintain the product correctly. Enclosed to these instructions for use you will find instructions for use for patients which have to be given to the patient and/or the care team.

For information reasons and for the safety of your patients, please note all information provided in these instructions for use including notes, tables and illustrations. Please note especially the safety instructions indicated by ATTENTION! CAUTION! WARNING! and DANGER! which are listed and explained in the following paragraph. Ignoring this information may lead to patient injuries and property damage.

4. Safety Instructions

4.1 Classification of the Safety Instructions

DANGER

Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.

WARNING!

→ Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.

CAUTION!

Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.

ATTENTION!

Important information about a possible situation which, if not avoided, leads to damage of the product.

All serious incidents according to Regulation (EU) 2017/745 which are related to the product have to be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

4.2 Structure of the Safety Instructions

The following example demonstrates the structure of the safety instructions:

Attention symbol	Signal word	ATTENTION!
	Type of danger	Mechanical load is too high!
\Box	Possible consequences for components	→ Wear of the joint parts.
V	Possible consequences for the patient	→ Maintenance intervals shorten.
	Measures	Explain to the patient the correct use of the system joint and possible dangers.

4.3 All Instructions for a Safe Handling of the P1 Foot Lifter Orthosis

There is detailed information to some of the safety instructions in the course of these instructions for use. Respect the following safety instructions to avoid patient injuries and property damage.

WARNING!

The orthosis is not adapted to the patient's shoe and foot!

Pressure points.

Adapt the orthosis to the patient's shoe and foot.

WARNING!

Spring bar is shortened too much!

Loss of the foot lifter.

Shorten the spring bar as described in these instructions for use.

WARNING!

Spring bar is too long!

Pressure points in the hollow of the knee.

Shorten the spring bar as described in these instructions for use.

CAUTION!

Orthosis is not cleaned after wearing it in salt water!

→ Corrosion of the riveted joints.

→ Skin irritations and impairment of wound healing.

Explain to the patient the correct use of the orthosis and possible dangers.

ATTENTION!

Carbon fibre sole is heated!

→ Material loses strength and can break.

Do not heat the carbon fibre sole.

ATTENTION!

Surface of the carbon fibre sole is ground!

The structure of the material is destroyed and the sole can burst.

Grind only the sole edge of the carbon fibre sole and not the surface.

ATTENTION!

Carbon fibre calf band is overheated!

→ Material loses strength and can break.
 Heat the carbon fibre calf band as described in these instructions for use.

5. Application

The P1 foot lifter orthosis is exclusively for use for orthotic fittings of the foot. It must be adapted by a professionally trained user.

6. Functionality

The P1 foot lifter orthosis is used for a drop foot. For cases of partial or total failure of the dorsiflexors, the P1 foot lifter orthosis holds the foot in a functional position of 90° (fig. 1).

Fig. 1

7. Scope of Delivery

The scope of delivery includes the following articles (fig. 2):

Description	Qty.
P1 foot lifter orthosis with accessories	1

8. Choose the Right Orthosis Size

Determine the patient's shoe size and choose the corresponding orthosis size.

Fig. 2

Shoe size	Sizes	Leg	Article number
35-37	Small	Left	PO1100-S/L
35-37	Small	Right	PO1100-S/R
38-40	Medium	Left	PO1100-M/L
38-40	Medium	Right	PO1100-M/R
41-43	Large	Left	PO1100-L/L
41-43	Large	Right	PO1100-L/R
44-46	X-Large	Left	PO1100-XL/L
44-46	X-Large	Right	PO1100-XL/R

9. Tools for Assembling the P1 Foot Lifter Orthosis

Tools
Hacksaw
Bending iron with rounded edges
Rivet snap
Hammer
Hot-air gun
Scissors

10. Adapting and Processing the P1 Foot Lifter Orthosis

WARNING!

The orthosis is not adapted to the patient's shoe and foot.

→ Pressure points.

Adapt the orthosis to the patient's shoe and foot.

Adapt the orthosis to the patient's shoe and foot and follow the next steps to ensure an optimal fit and functioning of the P1 foot lifter orthosis.

10.1 Carbon Fibre Sole

ATTENTION!

Carbon fibre sole is heated!

→ Material loses strength and can break.

Do not heat the carbon fibre sole.

ATTENTION!

Surface of the carbon fibre sole is ground!

→ The structure of the material is destroyed and the sole can burst.

Grind only the sole edge of the carbon fibre sole and not the surface.

- If the shoe has a removable insole, place the P1 foot lifter orthosis underneath (fig. 3). If the shoe has no removable insole, use the provided padding sole (see paragraph 10.6). If necessary, longitudinal and/or transverse arch supports can be additionally glued under the padding sole.
- 2. Adapt length and width of the carbon fibre sole to the patient's shoe. For this purpose, grind the edges (not the surface) of the carbon fibre sole on a drum sander with middle-grained abrasive.
- 3. Do not heat the carbon fibre sole. It is produced of thermosettingmatrix composites and is therefore not thermoformable.

Fig. 3

Note:

Respect the general safety instructions during grinding of carbon fibre materials (extraction, breathing mask, gloves, etc.).

10.2 Spring Bar

WARNING!

Spring bar is shortened too much!

→ Loss of the foot lifter.

Shorten the spring bar as described in these instructions for use.

WARNING!

Spring bar is too long!

→ Pressure points in the hollow of the knee.

Shorten the spring bar as described in these instructions for use.

- 1. After adapting the P1 foot lifter orthosis to the patient's shoe, let the patient try it on together with the shoe.
- 2. The spring bar should end at the maximum circumference of the calf (fig. 4). Shorten the spring bar with a hacksaw to the correct length and round the end with a suitable tool.
- 3. If you have to adapt the fitting of the spring bar in the heel area, use a bending iron with round edges. The carbon fibre sole is already shortened in the heel area and thus enables a precise fitting (fig. 5). This cut-out in the heel area can be compensated by the provided padding sole (see paragraph 10.6).
- 4. Preproduced shoes have anatomically shaped caps that are a little smaller in the middle than the spring bar of the P1 foot lifter orthosis. Therefore, some shoes can be damaged because of the pressure applied by the spring bar. To avoid this, cover the back of the spring bar with padding material.

Fig. 4

Fig. 5

10.3 Heat Shrink Tube

- Every orthosis is delivered with a heat shrink tube to cover the spring bar (fig. 6).
- Heat the shrink tube carefully from bottom to top using a hot-air gun (fig. 7). By that, the tube shrinks around the spring bar fitting perfectly and you achieve a visually appealing surface.
- Shorten the shrink tube at the proximal end, afterwards, so that you have enough space to mount the calf band (2 bores; fig. 8).

Fig. 6

Fig. 7

Fig. 8

Fig. 9

10.4 Calf Band

ATTENTION!
Carbon fibre calf band is overheated!

→ Material loses strength and can break.
Heat the carbon fibre calf band as described in these instructions for use.

- 1. Mount the carbon fibre calf band at the proximal end of the spring bar to achieve lateral quidance (fig. 9).
- If the calf band does not fit positively on the calf, the material can be adapted to it by thermoforming.
- 3. Heat the surface with a hot-air gun to 175°C with circling movements. Use our temperature marker (article no. ZM1000) to ensure the proper processing temperature. When heating, keep a distance of about 20cm between material and source of heat. The proper processing temperature is achieved after approx. 1–2 minutes. Now, form the heated material until it fits tightly.
- Fix the calf band on the spring bar using the delivered compression rivets when the calf band has cooled.

10.5 Hook and Loop Fastener and Pad

- The hook and loop fastener and pads are delivered with 6 selfadhesive hook dots. Those are needed to fix the hook and loop fastener and the pad to the calf band (fig. 10).
- Adhere three evenly spaced hook dots each on the outer and inner side of the calf band.
- 3. Fix the hook and loop fastener on the outer side and the calf pad on the inner side.
- Pull the sliding pad for the tibia over the strap of the hook and loop fastener.

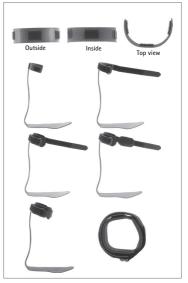
Fig. 10

10.6 Padding Sole

- 1. Place the carbon fibre sole on the (white) back side of the padding sole and make sure that the protrusion at the bottom of the padding sole fits tightly to the heel area of the carbon fibre sole (fig. 11).
- Cut the padding sole with scissors to the fitting size. But make sure to have enough material left to cover the edges of the carbon fibre sole.
- 3. Fix the padding sole on the carbon fibre sole using the double-sided adhesive glue dots (fig. 12).

Fig. 11

Fig. 12

Note:

Make sure that patients with diabetes wear shoes that provide enough volume and, additionally, place a custom-made foot orthotic suitable for diabetic patients on the padding sole.

11. Putting the P1 Foot Lifter Orthosis on

- 1. Place the P1 foot lifter orthosis in the shoe (see paragraph 10.1, point 1).
- Open the shoe as wide as possible to avoid excessive bending of the spring bar when putting the shoe on (fig. 13) to avoid any damages to the orthosis.
- 3. Let the patient put the shoe on and tie it.
- Close the hook and loop fastener of the P1 foot lifter orthosis. The hook and loop fastener has to fit tightly to the leg but must not apply pressure to muscles and soft parts.
- 5. Check if the orthosis fits well and functions properly.

Fia. 13

12. Cleaning

CAUTION!

Orthosis is not cleaned after wearing it in salt water!

- Corrosion of the riveted joints.
- ⇒ Skin irritations and impairment of wound healing.

Explain to the patient the correct use of the orthosis and possible dangers.

The durability of the P1 foot lifter orthosis requires proper handling. Regularly clean the pads and the hook and loop fastener with a gentle detergent. Rinse it with clear water after wearing the P1 foot lifter orthosis in salt water.

13. Spare Parts (see table and fig. 14)

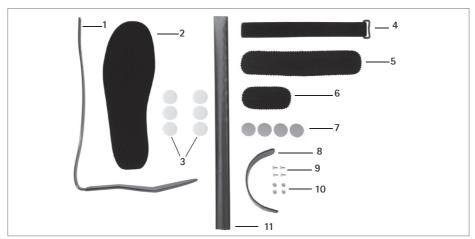
Fig. 14

Item	Article number	Description
1	P00110-*/**	P1 carbon fibre foot lifter orthosis
2	P00300-**	P1 padding sole with glue dots
3	KV0020-22/06	Hook dots, self-adhesive, transparent, 6 piece
4-6	KV2004-L450	Hook and loop fastener with loop and pad, 25mm wide, 450mm long
4-6	KV2004-L500	Hook and loop fastener with loop and pad, 25mm wide, 500mm long
4-6	KV2004-L550	Hook and loop fastener with loop and pad, 25mm wide, 550mm long
7	KL3000	Glue dot, double-sided adhesive, round, $\emptyset = 30$ mm
8	P00101-M	P1 carbon fibre calf band, calf circumference 29-36cm
8	P00101-XL	P1 carbon fibre calf band, calf circumference 37-44cm
9	NI2084-L09	Lower part for compression rivet 4 x 9mm
10	NI2184-L04	Upper part for compression rivet 4 x 4mm
11	P00200	P1 heat shrink tube

^{*} size (e.g. L for large - P00110-L/**) ** side (e.g. L for left - P00110-*/L)

Notizen/Notes		

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The information in these instructions for use is valid at the date of printing. The contained product information serves as a quideline. Subject to technical modifications.

All rights, particularly the distribution, copy and translation of these instructions for use or any part of it, in paper or as electronic document, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädietechnischen Systemen mbH. Reprints, copies and any other electronic reproduction, even partial, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädietechnischen Systemen mbH.

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The quality standard of the FIOR & GENTZ company has been controlled and certified by an independent certification organisation

according to the international standards ISO 9001 and ISO 13485.